

# TECHLIFT 500

## Adjustment of overload

Remove the six screws on the cover (Torx 15 M5 x 8), remove the cover (see Figure 1)

Release the screws on the plate to the switch( hexagon socket head screws **M4**),move the plate to the back of the cover( see figure 2)

Test the lift and adjust if necessary, when the switch is activated a signal sounds and the small red light on top of the cover is on.

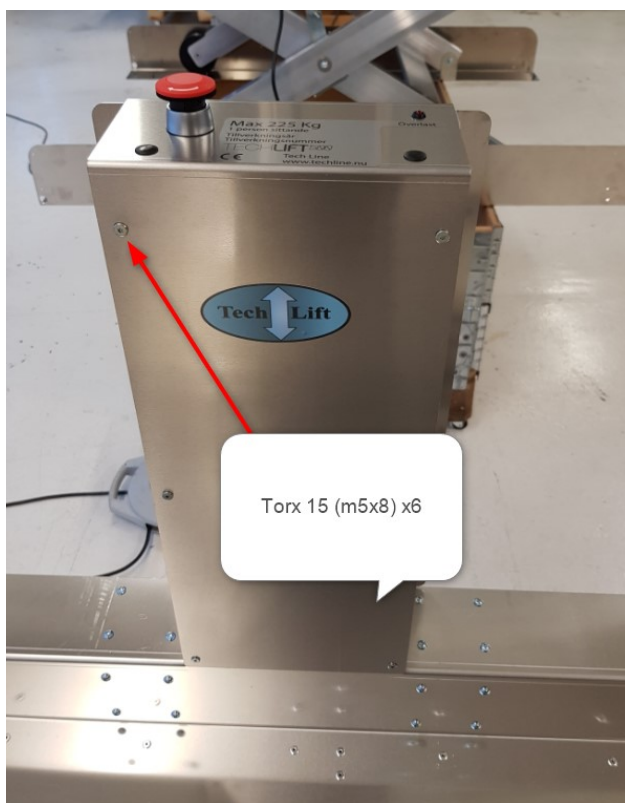


Figure 1

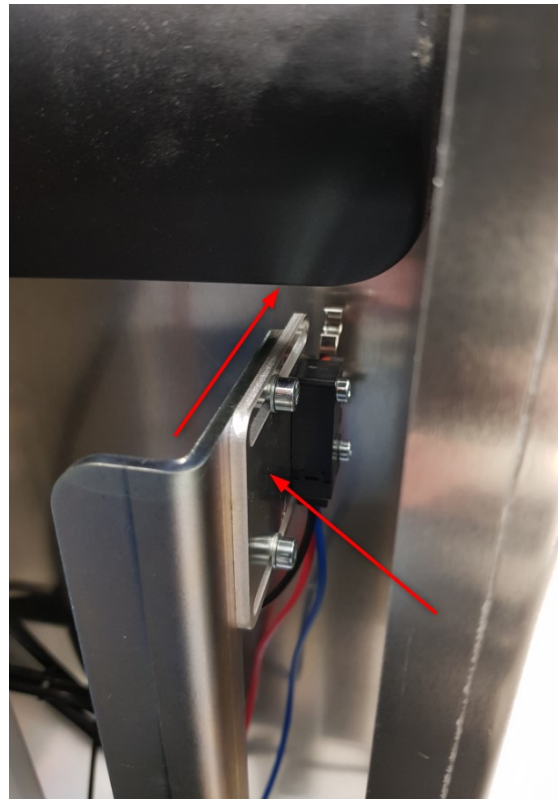


Figure 2

TECHLINE

Manufacturer TECH LINE Kyrkbacken 2 SE- 507 90 Bråmhult

Tel: + 46 (0) 709 837972 Mob: +46 (0) 705 88 96 19

E-Post: [stefan@htu-center.se](mailto:stefan@htu-center.se)

Webb: [www.techline.nu](http://www.techline.nu)